

Description


Amplified inductive pick-up coil for use in hearing instruments. The coil has a solid gold colored coating for excellent environmental protection and extra protection against mechanical stress. The integrated pre-amplifier features a high EMI suppression and a higher-order low frequency roll off for LF noise (50/60 Hz) attenuation. The output stage is equivalent to the one found in Sonion microphones with CMOS ICs (e.g. 6000, 96/9700 and 1M000 series), and allows direct switching between microphone and coil. The response curve is flat which ensures good matching of the telecoil and microphone response.




Absolute maximum ratings

Parameters	Symbol	Min	Max	Unit
Supply voltage	Vdd	0.9	3.6	VDC
Operating temperature	Tw	-10	50	°C
Storage temperature	Ts	-40	80	°C
ESD protection level	Vesd	3		kV

ESD protection level: Class 2 according to MIL-STD-750D, test method 1020,2. Apply protection in accordance with IEC 61340-5-1 and 61340-5-2.

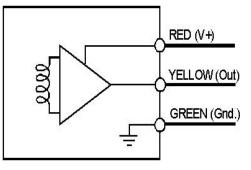

Product drawing - Dimensions in mm [inch]

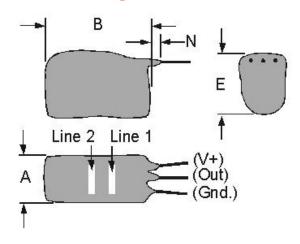

Mechanical data

Dimensions	in mm	in inch
Dimension A max.	2.39	0.094
Dimension B max.	5.52	0.217
Dimension E max.	3.1	0.122
Dimension N max.	2	0.079

Color code	
Line 1	Blue
Line 2	Grey

Lead wires

Parameters	Supply lead (V+)	Signal lead (Out)	Ground lead (Gnd)	
Diameter in mm	0.16	0.16	0.16	
Plating	Silver	Silver	Silver	
Color	Red	Yellow	Green	
Length in mm *	30 ± 1	30 ± 1	30 ± 1	
Strip/tin free end	None	None	None	
* Length measured from flange				

Assembly notes

None

Sonion reserves the right to make changes at any time to improve reliability, function or design, in order to provide the best product possible. Receivers series of this type can produce very high sound pressure levels. When such receivers are applied in hearing instruments or other communications equipment special attention should be paid to this capacity in order to prevent possible hearing damage.

DK: +45 4630 6666 USA: +1 952 543 8300 PRC: +86 512 6832 3401 NL: +31 20 6068 100



Version date

Data Sheet

Amplified telecoil TA 20 AE 21



Specifications

Measuring conditions (unless stated otherwise): supply voltage = 1 VDC, frequency = 1 kHz, temperature = 23°C (73.4F), H = 0.1 A/m and load = 10 pF/200 kOhm. All parameters were measured according their specific MQP.

Parameters		Symbol	Min	Тур	Max	Unit	Comments
Supply voltage		Vdd	0.9	1	1.5	VDC	
Supply current		ldd	40	55	90	μA	Vdd: 0.9 V-1.5 V
DC output voltage	DC output voltage		450	580	800	mV	Vdd: 0.9 V-1.5 V
Output impedance	Output impedance		1700	3000	4500	Ohm	Vdd: 0.9 V-1.5 V
	@ 200 Hz	S200	-69.5	-67	-64.5	I.D.	
Sensitivity	@ 1 Khz	S1k	-51.5	-49.5	-47.5	dB re. 1V/0.1Am- ¹	
	@ 5 kHz	S5k	-49	-46.5	-44	1 0/0.1/4111-	
	@ 200 Hz	S200	-72	-69.5	-64.5		
Sensitivity	@ 1 kHz	S1k	-54	-52	-50	dB re. 1V/0.1Am- ¹	10 kOhm load
	@ 5 kHz	S5k	-51.5	-49	-46.5	1 V/O. IAIII-	
1st High pass cuto	1st High pass cutoff frequency		175	217	270	Hz	
2nd High pass cut	2nd High pass cutoff frequency		0.85	1.05	1.3	kHz	S@5 kHz - 3 dB
1st Low pass cuto	1st Low pass cutoff frequency		15.8	19.8	24	kHz	S@5 kHz - 3 dB
Equivalent input noise ¹		Ni		30	32.5	dB MFL *	0.1-10 kHz (Awgt)
Maximum input signal ²		Mi		87.5		dB MFL *	THD < 1%
Dynamic range		DR		-57.5		dB	
Total harmonic distortion		THD		0.3	1	%	Vo = 10 mV rms
Power supply rejection ³		PSR	16	20		dB	0.1-10 Khz
Equivalent EMI	@ 0.9 GHz	Nemi0.9		35		dB MFL *	Near-field
noise ¹	@ 1.9 GHz	Nemi1.9		77		dB MFL*	measurement
50Hz suppression re. 2 kHz level		dB		50		dB	

Notes

Sonion reserves the right to make changes at any time to improve reliability, function or design, in order to provide the best product possible. Receivers series of this type can produce very high sound pressure levels. When such receivers are applied in hearing instruments or other communications equipment special attention should be paid to this capacity in order to prevent possible hearing damage.

DK: +45 4630 6666 USA: +1 952 543 8300 PRC: +86 512 6832 3401 NL: +31 20 6068 100



 $^{^*}$ 0 dB MFL (Magnetic Field Level) - 0 dB SPL (70 dB SPL- 100 mA/m is the typical conversation level) 0 dB MFL - 31.6 μ A/m = Hr. Hi [MFL] = Vo [dB re 1V] - S [dB re 1V/Am-¹] - Hr [dB re 1Am-¹] = Vo [dB re 1V] - S [dB re 1V/0.1Am-¹] + 70

¹ A-weighted input refered noise voltage

² With maximum THD of 1%


³ Power supply rejection (PSR) is expressed as the maximum of inverse function of the small-signal voltage gain from the positive power supply line to the output of the amplifier in the frequency range 0.1-10kHz.

Data Sheet


Amplified telecoil TA 20 AE 21



Typical response curve

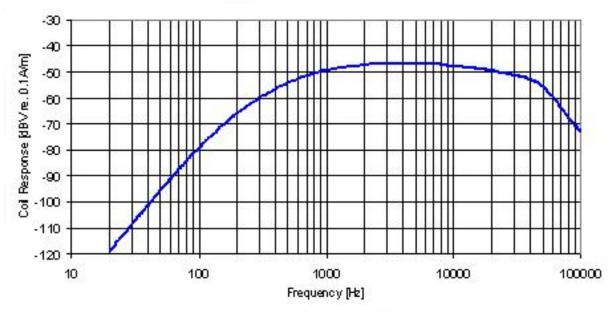

Cutoff frequency

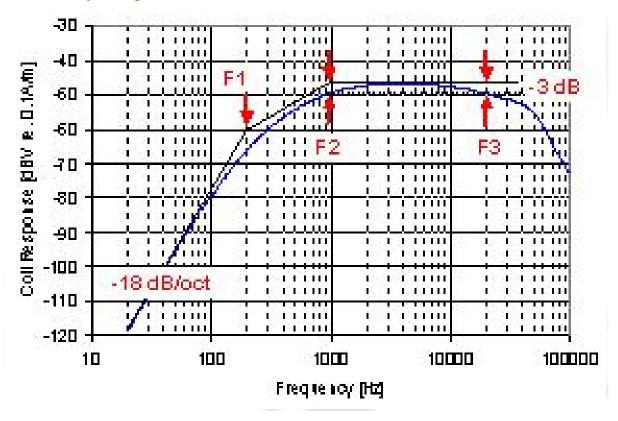

Sonion reserves the right to make changes at any time to improve reliability, function or design, in order to provide the best product possible. Receivers series of this type can produce very high sound pressure levels. When such receivers are applied in hearing instruments or other communications equipment special attention should be paid to this capacity in order to prevent possible hearing damage.

+45 4630 6666 USA: +1 952 543 8300 PRC: +86 512 6832 3401 NL: +31 20 6068 100

